GB Because of the opportunity to work at three different temperature levels, this *gumshade* Kit pre-eminently covers the demand for comprehensive gingiva substitution with implant cases & supra-structures.

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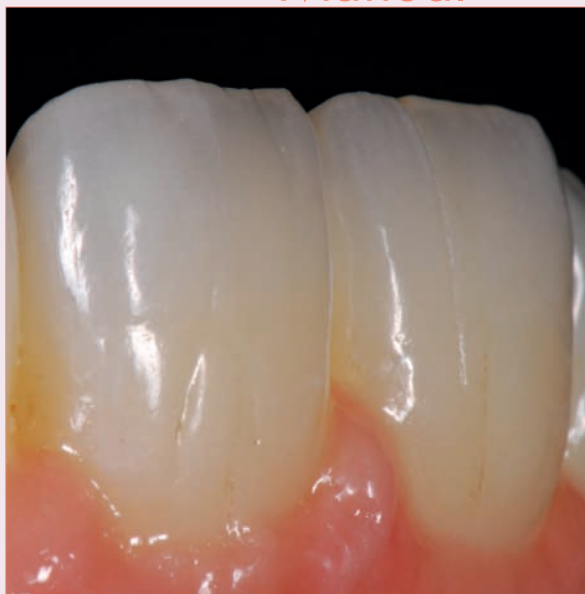
Instructions for use of Antagon Interaction gumshades

Indication: The range of Interaction *gumshades* has been developed to create natural-looking gum replacements, making aesthetic correction of large interdental gaps and tooth cervices for bridges possible. Gum replacements for surface implants can now be made to look as natural as possible.

Contraindications: Do not use in combination with Sakura Interaction and Carrara Interaction materials. Do not use if the patient is known to be allergic to one of the components.

Product range:

- paste opaque gum*: procedure and properties identical to regular *paste opaques*.
- 211 – 217: procedure and properties identical to regular *opaque dentin* and *dentin* materials. The *opaque dentins* are numbered 211 – 213 and the *dentins* 214 – 217.
- margin gum* and *margin correction*: procedure and properties identical to products from the *margin* ceramic range.

Application: **paste opaque gum**

A pink *paste opaque gum* is available for an optimum aesthetic result. This *paste opaque* can be applied either separately or as part of the regular *paste opaque* procedure.

- Apply the paste and fire it in accordance with the usual *paste opaque* firing programme (see table).

Drying	Start temp.	Vacuum	Rate of heat increase
7 min.	400°C	yes	60°C/min.
Final temp.	Holdtime	Vacuum	Surface appearance
915°C	2 min.	1 min.	silky matt, shiny

margin gum

If desired, gum-coloured material can be applied during a *margin* firing phase.*

- Mix the *margin gum* material with *margin liquid* into a cream-like consistency.
- Apply the paste in layers.
- Condense thoroughly and blow dry with hair-dryer if necessary.
- Remove the object from the model and fire in accordance with the regular Antagon Interaction shoulder preparation firing phase (see table).

Drying	Start temp.	Vacuum	Rate of heat increase
8-10 min.	500°C	yes	60°C/min.
Final temp.	Holdtime	Vacuum	Surface appearance
915°C	3 min.	2 min.	silky matt

dentin gum

- Use the Shadeguide to select the desired colour shade.
- Mix the selected *gumshade* material(s) with *carving liquid* into a cream-like consistency and apply in layers.
- Condense thoroughly and remove the object from the model.
- Fire in accordance with the regular Antagon Interaction *dentin* firing chart (see table).

Drying	Start temp.	Vacuum	Rate of heat increase
5-7 min.	500°C	yes	60°C/min.
Final temp.	Holdtime	Vacuum	Surface appearance
895°C	1-2 min.	1 min.	silky matt

margin correction

- For the *margin correction* firing please consult above listed working method and fire according firing table below.*

Drying	Start temp.	Vacuum	Rate of heat increase
6 min.	500°C	yes	60°C/min.
Final temp.	Holdtime	Vacuum	Surface appearance
850°C	2 min.	no	silky matt

* Please also consult the Antagon Interaction *margin* instructions for use.

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Elephant Dental B.V.