Carrara Shade paints

NL

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GB

Shade paint program for StratoPlus / StratoPress:

Dry 4 min.	Starttemp. 450°C	Vacuüm Start 450°C	
Rate of increase 55–80°C	Final temp. 830°C	Vacuüm Off 830°C	Hold time 1–2 min. (without vac.)

Indication: • Dental ceramic for the application of colour characteristics to the material surface of ceramic press and ceramic-faced dental crowns and bridges. • Suitable for use in combination with Carrara Volumia, Carrara Interaction and other ceramics with a similar range of indications. **Contraindication:** • Not suitable for use in combination with ceramic systems other than those listed under 'Indication'. • Not suitable for use as an intermediate layer between ceramic masses. **Processing:** • The CSP1–CSP9 Shade paints can be used for easy application of natural colour effects and anomalies and easy colouring of fissures. • These Shade paints can be used to enhance or optimise the natural colour effect of the 16V colours (see Full-Contour Colour Table). • Before the glaze firing process, clean the preparation thoroughly using a dental steam cleaner. • Mix the Shade paint powder using Paint Liquid until a homogeneous mass is obtained. • Apply a thin, even layer of the mixed Shade paint colour to the selected locations. • Fire according to the data in the firing table. • Repeat the process for more intensive colour effects.

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Art.nr.

Antagon & Carrara Interaction Glaze	3,0g	484203071
Paint Liquid	20ml	244004130

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Elephant Dental B.V.