## E

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**Samenstelling (in massapercentage)**

Kobalt ..... 61,1%
Chroom ..... 32,0%
Molybdeen ..... 5,5%
Si, Mn < 1,0%
De legering Vi-Comp is nikkel en beryllium vrij.

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©2006 DENTSPLY International Inc.  
All rights reserved.  
Printed in U.S.A.

**Indications**

Vi-Comp Alloy is a cobalt-based dental ceramic alloy, intended for metal ceramic or full metal dental restorations.

**Contraindications**

- Vi-Comp Alloy is contraindicated for patients and users with a history of sensitivity to cobalt, chromium, and molybdenum.

**Warnings**

- Prolonged exposure to alloy dust and/or fumes may lead to lung irritation, and/or pulmonary complications. Use appropriate engineering controls to limit exposure. For excessive inhalation of dust or fumes, seek medical advice.
- This alloy contains chromium. Some compounds of this element are potential carcinogens. Advise your health care provider of exposure to this element.
- This alloy contains elements that are known to the State of California to be carcinogenic.

**Precautions**

- When melting, grinding, or polishing alloys, use adequate ventilation, vacuum systems, protective eyewear, and protective masks and clothing.
- Check with the porcelain manufacturer to assure matched average thermal expansion of selected porcelain and Vi-Comp Alloy.

**Adverse Reactions**

- Exposure to alloy dust or fumes may cause eye irritation and/or respiratory complications.
- Certain components of alloys are potential carcinogens. See Warnings.

## STEP BY STEP INSTRUCTIONS

**WAX-UP AND SPRUING:**

Use a die spacer and wax copings to a minimum of 0.4 mm. Indirect spruing for multiple units and direct for single units; use 10 gauge (2.6mm) sprue for copings; 8 gauge (3.2mm) sprus for bulky crowns and pontics. Sprus should be 6.5 to 9mm in length. Use 5 gauge (4.5mm) runner bars. The space between the runner bar and pattern should be 2-4mm.

**INVESTMENT:**

Use a high heat phosphate investment such as Vi-Comp® Investment (ReOrder# N251500).

**BURNOUT INSTRUCTIONS:**

Bench set molds for one hour. Place molds in a cold furnace and heat molds to 1800°F (980°C) following the manufacturer’s recommendations. Hold for one hour.

**TORCH CASTING:**

Use a separate high heat crucible for melting Vi-Comp Alloy. Use propane oxygen or oxygen acetylene multi-orifice torch. Wind casting machine one more turn than used for gold alloys.
**Note: Do NOT break oxide layer. The casting temperature is reached when the molten mass rolls or waves from pressure of flame.**

**INDUCTION CASTING:**

DO NOT USE CARBON LINER IN A CRUCIBLE. A preheated crucible will minimize the amount of metal left in the crucible after casting.

**Machine Settings:**

Casting Temperature Setting: ..... 2800°F (1537°C)
Soak Timer: ..... 15 seconds
Pyrometer position: ..... Back (focused on the alloy)
Acceleration reducer: ..... 10-30

**SURFACE PREPARATION:**

Grind all alloy surfaces, which are to receive porcelain, using aluminum oxide stones. Blast the surface using non-recycled 125 micron dental grade aluminum oxide, max 3 bar (44 psi). The framework should be steam cleaned or ultrasonically cleaned in distilled water. (DO NOT use soap, acids or solvents).

**OXIDATION:**

- Heat the object from 1400°F (760°C) to 1800°F (980°C) at a heat rate of 120°F (50°C) per minute without vacuum.
- Remove the object immediately from the furnace.
- After the oxidation the object shows a smooth, olive-green oxide layer
- Wash bake: apply a thin layer of opaque slurry or paste opaque, process according to the instructions of the porcelain manufacturer, the final temperature should be at least 1800°F (980°C), with a hold time of 1-2 minutes.

**OPAQUE AND BODY PORCELAIN BUILD-UP:**

Apply and fire per manufacturer’s instructions

**SOLDERING (Recommended Solders & Fluxes):**

Pre-Soldering: ..... Use a gas-oxygen torch.

Pre-Solder: .....ReOrder# N764300
White Bake-On Pre-Solder

Pre-Soldering Flux: .....ReOrder# N200122
Pre-Porcelain Soldering Flux
High Fusing

Post Soldering: .....ReOrder# N086100
583 Fine Low Fusing Gold Solder

Post Soldering Flux: .....ReOrder# N200224
Post-Porcelain Soldering Flux

**PHYSICAL & MECHANICAL PROPERTIES:**

Vickers Hardness (after firing) ..... 320 HV5
Melting Range ..... 2480-2550°F (1360-1398°C)
Casting Temperature ..... 2800°F (1540°C)
0.2% Yield Strength ..... 64,900 psi (448 MPa)
Tensile Strength ..... 100,800 psi (695 MPa)
Elastic Modulus ..... 25,381 kpsi (175 GPa)
Color ..... Gray
Percent Elongation Range ..... 7.7%
Density ..... 8.3 g/cc
Average Coefficient of Thermal Expansion:..... (25-500°C) (14.25 x 10<sup>-6</sup>/°C)

**Composition (% by mass):**

Cobalt (Co) ..... 61.1%
Chromium (Cr) ..... 32.0%
Molybdenum (Mo) ..... 5.5%
Si, Mn < 1.0%
Nickel and Beryllium Free Alloy

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