## User instructions

INTERACTION® SAKURA

# Spray Liner

Low fusing ceramic Spray Liner  
Type I dental ceramic

Rx only  
ISO 6872

Elephant Dental B.V.  
Verlengde Lageweg 10  
1628 PM Hoorn, The Netherlands  
Tel: +31 229 25 90 00  
Fax: +31 229 25 90 99  
E-mail: info@elephant.nl  
www.elephant-dental.com

Elephant Hoorn Holland

Made in Holland

Base shade and fluorescence in one firing  
Glaze multiple units in seconds  
Reduce labor cost  
Increase production efficiency  
Won't puddle at margins

CTE = 9,7 µm/m.K (25°C-Tg)  
Tg = 480°C

## Precautions

**Warning: Contents under pressure:** Avoid prolonged exposure to sunlight or heat from radiators, ovens or other heat sources. Exposure to temperatures above 50°C/120°F may cause can to burst and cause injury. Do not puncture, incinerate or place in trash compactor.

**Disposal of empty cans:** Cans should be disposed of in a manner consistent with local regulations.

**Vapor harmful:** Use with adequate ventilation and local dust collector with fume extractor. Intentional misuse deliberately concentrating and/or inhaling contents may be harmful or fatal.

R12 : Extremely flammable.  
R36/37: Irritating to eyes and respiratory system.  
S2 : Keep out of the reach of children.  
S9 : Keep container in a well-ventilated place.  
S16 : Keep away from sources of ignition – No smoking.  
S24/25: Avoid contact with skin and eyes.  
S26 : In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

**First aid:** In case of eye contact, see S26. In case of respiratory difficulty, move to fresh air and call physician. If swallowed, get medical attention immediately. Consult MSDS for more information.

**Contents:** Ceramic powder, Isopropyl alcohol, hydrocarbon propellant.

Net Weight: 5.08 oz./144 g

Powder Weight: 0.79 oz./22.5 g

Dental Ceramic Aerosol Packaging and Patent By Enamelite LLC, USA # 7,018,465.

## Spray Liner instructions

**Indication:** Use on zircon oxide sub-structures in combination with Interaction® Sakura ceramic.

**Contraindication:** Do not use in combination with other (metal) ceramics. Do not use on the ceramic surface of the restoration. If patients are known to be allergic to any of the components, the material should not be applied.

### Colourtable

Spray Liner Medium															
Spray Liner Light										Spray Liner Dark					
B1	A1	B2	A2	C1	D2	A3	D3	C2	B3	D4	A3,5	B4	C3	A4	C4

- Shake well until mixingball moves freely inside and afterwards keep on shaking for at least 20 seconds. Repeat this procedure everytime before you start spraying.
- Make sure the sub-structure is dry before spraying.
- Spray the liner from approximately 6 to 8 inches/15 to 20 cm over the restoration/s.
- Spray multiple units simultaneously when placed on ceramic pillow tray. Be sure to shake can often when spraying multiple units.
- If liner should settle at the inside of a restoration/s, once the liner dries, remove with a small short hair bristle brush.
- Firing schedule **Spray Liner** in combination with **Interaction® Sakura**:  
Pre-dry : 1 minute  
Start temp. : 450°C/842°F  
Heating : 60°C/140°F/min.  
High temp. : 850°C/1562°F  
Vacuumstart : 450°C/842°F  
Hold time : 1 min. with and 1 min. without vacuum
- For optimal product performance you should turn can upside down and depress spray button between applications. This will clear the valve of any trapped and unused liner that might cause a propellant leak from can.

Single units spray yield approximately 120 units per can.

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