User instructions

ANTAGON & CARRARA INTERACTION

Spray Glaze

Low fusing ceramic spray glaze
Type I Dental ceramic

Rx only
ISO 6872

Verlengde Lageweg 10
1628 PM Hoorn, The Netherlands
Tel: +31 229 25 90 00
Fax: +31 229 25 90 99
E-mail: info@elephant.nl
www.elephant-dental.com

Elephant Dental B.V.

Made in Holland

Stain and glaze in one firing
Glaze multiple units in seconds
Reduce labor cost
Increase production efficiency
Maintain detailed stain characterizations
Won't puddle at margins

Precautions

Warning: Contents under pressure: Avoid prolonged exposure to sunlight or heat from radiators, ovens or other heat sources. Exposure to temperatures above 50°C/120°F may cause can to burst and cause injury. Do not puncture, incinerate or place in trash compactor.

Disposal of empty cans: Units should be disposed of in a manner consistent with local regulations.

Vapor harmful: Use with adequate ventilation and local dust collector with fume extractor. Intentional misuse deliberately concentrating and/or inhaling contents may be harmful or fatal.

R12 : Extremely flammable.
R36/37 : Irritating to eyes and respiratory system.
S2 : Keep out of the reach of children.
S9 : Keep container in a well-ventilated place.
S16 : Keep away from sources of ignition – No smoking.
S24/25 : Avoid contact with skin and eyes.
S26 : In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

First aid: In case of eye contact, see S26. In case of respiratory difficulty, move to fresh air and call physician. If swallowed, get medical attention immediately. Consult MSDS for more information.

Contents: Ceramic porcelain powders, Isopropyl alcohol, hydrocarbon propellant.

Net Weight: 5.78 oz./164 g
Powder Weight: 0.79 oz./22.5 g

Dental Ceramic Aerosol Packaging and Patent by Enamelite LLC, USA # 7,018,465.

Spray Glaze instructions

Indication: Use on the ceramic surface of porcelain fused to metal restorations made with Antagon or Carrara Interaction or pressed with Carrara Volumia (apply 2 times and fire 2 times). (Carrara Press Inlay, apply 2 times and fire 2 times according to special high temp: 750-760°C / 1382-1400°F).
Contraindication: Do not use for patients known to be allergic to any of the components.

- Shake well until mixing ball moves freely inside.
- Prepare restoration/s as customary for staining and glazing.
- Spray glaze approximately 10 to 12 inches/25 to 30 cm from restoration/s over dried and unfired stains. For optimum performance use short bursts while shaking/agitating can between sprays on buccal, lingual, mesial, distal and occlusal surfaces.
- Spray multiple units simultaneously when placed on ceramic pillow tray. Be sure to shake can often when spraying multiple units.
- Minor correction repairs may be applied before or after spraying Antagon & Carrara Interaction glaze on dental restoration/s. It will not discolour add-on.
- Firing Schedule for **Carrara Interaction** and **Carrara Volumia:**
Pre-dry : 4 minutes
Start Temp : 450°C/840°F
Heating : 55-80°C/130-175°F/min.
High Temp : 830°C/1525°F
Vacuum : None
Hold Time : 1 minute
- Firing Schedule for **Antagon Interaction:**
Start Temp : 500°C/932°F
High Temp : 885°C/1625°F
All other instruction see above.
- If glaze should settle on the underside or on metal surfaces of a restoration/s, once the glaze dries, remove with a small short hair bristle brush.
- For optimal product performance you should turn can upside down and depress spray button between applications. This will clear the valve of any trapped of unused glaze that might cause a propellant leak from can.

Single units spray yield approximately 120 units per can.

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