Carrara Shade paints

NL

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GB

Shade paint program for StratoPlus / StratoPress:

Dry 4 min.	Starttemp. 450°C	Vacuüm Start 450°C	
Rate of increase 55–80°C	Final temp. 830°C	Vacuüm Off 830°C	Hold time 1–2 min. (without vac.)

Indication: • Dental ceramic for the application of colour characteristics to the material surface of ceramic press and ceramic-faced dental crowns and bridges. • Suitable for use in combination with Carrara Volumia, Carrara Interaction and other ceramics with a similar range of indications. **Contraindication:** • Not suitable for use in combination with ceramic systems other than those listed under 'Indication'. • Not suitable for use as an intermediate layer between ceramic masses. **Processing:** • The CSP1–CSP9 Shade paints can be used for easy application of natural colour effects and anomalies and easy colouring of fissures. • These Shade paints can be used to enhance or optimise the natural colour effect of the 16V colours (see Full-Contour Colour Table). • Before the glaze firing process, clean the preparation thoroughly using a dental steam cleaner. • Mix the Shade paint powder using Paint Liquid until a homogeneous mass is obtained. • Apply a thin, even layer of the mixed Shade paint colour to the selected locations. • Fire according to the data in the firing table. • Repeat the process for more intensive colour effects.

Indikation	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
Liner	A1	A2	A3	A3,5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
Transpa Dentin Pellet	TD1	TD1	TD2	TD2	TD3	TD4	TD4	TD5	TD5	TD1	TD2	TD2	TD3	TD1	TD6	TD6
Carrara Shade paint	1	2	2	2	2	4	5	4	4	5	6	7	7	3	8	9

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Antagon & Carrara Interaction Glaze 3,0g 484203071

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Elephant Dental B.V.