## Laser Welding Wire User Manual

**Indication:** Filler material for laser welding of CoCr dental alloys, which are suitable for laser welding.

**Contraindication:** Do not use in cases of known allergy to any of the ingredients.  
Not to be used in combination with the customary flame or furnace soldering methods.

**Precautions:** When using a laser welder, the safety precautions provided by the manufacturer must be observed.  
To prevent any unnecessary material blending, the laser welding wires to be used should be of identical alloys.

**Choice of material:** Laser welding wires are used as laser welding filler material.

### General Recommendation for Laser Welding Wires:

Laser welding wires should not be used as solder with the usual flame or furnace soldering method.

### Processing Recommendations:

- Observe the manufacturer's recommendations for the device in question.
- Set the laser welder to the specific value required for the type of alloy used.
- Test the basic settings first on a piece of casting sprue whose surface has been pretreated with a silicone point and/or has been sandblast. Adjust the laser parameters as necessary.
- ! Reflective surfaces cannot be laser welded. The metallic area MUST be matt!
- Before laser welding, cast metal products should undergo heat treatment to prevent any material stresses
- Use of a protective gas prevents oxidation of the welding environment.
- Preferably use X-shaped laser welds.
- Build up the laser shots crosswise.
- We recommend that the object undergoes another heat treatment after laser welding to avoid material stress.

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