# Sakura Shade paints

NL

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GB

Shade paint firing program for StratoPlus / StratoPress:

Dry 4 min.	Starttemp. 450°C	Vacuum Start 450°C	Rate of increase 60°C
Final temp. 810°C	Vacuum Off 810°C	Hold time 1-2 min. (without vac.)	Cooling 0 min.

**Indication:** • Dental ceramic for the application of colour characteristics to the material surface of ceramic press and ceramic-faced dental crowns and bridges. • Suitable for use in combination with *Sakura Volumia*, *Sakura Interaction* and other ceramics with a similar range of indications. **Contraindication:** • Not suitable for use in combination with ceramics other than those listed under 'Indication'. • Not suitable for use as an intermediate layer between ceramic masses. **Processing:** These *Shade paints* can be used to enhance or optimise the natural colour effect of the 16V colours (see Full-Contour Colour Table). The SSP1 - SSP9 *Shade paints* can be used for easy application of natural colour effects and anomalies and easy colouring of fissures. • Before the glaze firing process, clean the preparation thoroughly by using a dental steam cleaner. • Mix the *Shade paint* powder using *Paint Liquid* until a homogeneous mass is obtained. • Apply a thin, even layer of the mixed *Shade paint* colour to the selected locations. • Fire according to the data in the firing table. • Repeat the process for more intensive colour effects.

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Elephant Dental B.V.