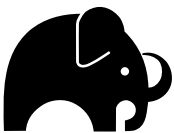
# Carrara Universal DL Investment

*pressen, pressing, persen, pressage, pressatura, prensado*

(2x)60 - 100 g

Elephant

dental health products

CUDL 1003

# Inhaltsangabe

## Index

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# Protocole

## Indice

## Indice

**D**

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**FR****GB**

- 3 Product profile
- 4 Physical properties
- 5 Remarks
- 6 Investing
- 17 Pre-heating
- 19 Finishing
- 20 Tips & tricks CP200
- 22 Tips & tricks Ringformer Paper

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**Product profile:**

Carrara Universal DL Investment is a universal phosphate- bound, graphite- free dustless investment for the casting of dental precious alloys and pressing ceramics. The investment powder comes packed in sachets of 60, 100, 150 g or bucket of 6 kg. The associated Carrara Universal DL Investment Liquid is a concentrated mixing liquid, which, when diluted with distilled water, guarantees perfect workability and fit.

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GB

**Physical properties:**

Mixing ratio: 100 g/24 ml - 120 g/28,8 ml  
 Mixing time: 60-120 sec. under vacuum  
 Processing time: 7 min. at 21°C/70°F  
 Expansion during setting: 0.4% (60%)  
 (Carrara Universal DL Investment Liquid, without Liner)  
 Thermal expansion: 1,1% (at 500°C/932°F)  
 Compressive strength: 6,7 MPa

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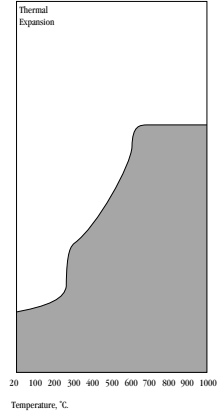
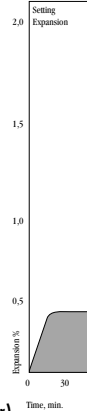
**Remarks:**

1. The working time of Carrara Universal DL Investment can be increased by keeping the liquid in the refrigerator. Close the bottle after use. When frozen or evaporated, the liquid will crystallize on the bottom, and the liquid can no longer be used
2. STORAGE: Store powder in a dry and cool place (temp.: 8-25°C/46-77°F).
3. Use the mixing bowl for phosphate-bonded investments only.
4. Carrara Universal DL Investment contains silica. This release is reduced to a minimum because of the patented treatment. Avoid inhalation of dust.
5. Ensure adequate extraction and ventilation during the burning-out process.

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GB

- Moisten the wax-up objects with an alcohol-free surface-tension-reducing wetting agent.

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GB

- Treat the Silicon Muffle Ring with silicon spray (oil free), which facilitates removal of the ring from the investment material.

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GB

- Mix powder and liquid with a spatula for 10-15 sec.
- Mix the Carrara Universal DL Investment under vacuum for 60-120 seconds (100 g powder with 24 ml diluted liquid).
- The recommended dilution ratio of Carrara Universal DL Investment Liquid to distilled water is 55-45% to 60-40%.

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GB

- Immediately after filling, attach the Muffle Topshaper with a twisting movement.
- After filling, allow the Carrara Universal DL Investment to set for 15 minutes.
- If the hardening time is exceeded, the muffle can no longer be heated quickly. It should then be heated in phases (slow process).

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GB

- Shortly before removing the Muffler Base, slightly roughen the bottom of the muffler with sandpaper (120 grit). Make certain that the bottom remains even.

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GB

- Remove the Muffle Base from the muffler using a slight twisting movement.
- Remove the upright investment edge.
- If necessary remove any investment material residue with compressed air.

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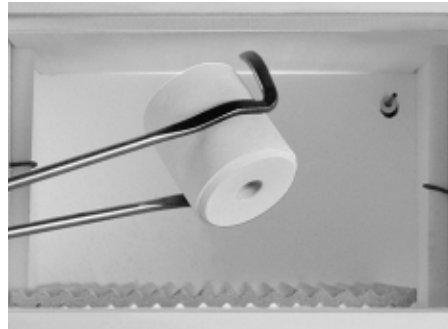
GB

**Direct heating (quick process):**

- Place the muffle in a pre-heated furnace at a temperature of 600°C/1112°F.
- The muffle should stand in the furnace at a temperature of 600°C/1112°F for 10 minutes.
- Allow the temperature to rise to 950°C/1742°F over a period of 23 minutes.

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- GB**
- Devest the castings by sandblasting with glas beads (50 µm).
  - Maximum pressure 2.5 bar for Carrara Press Core.
  - Maximum pressure 2.0 bar for Carrara Press Inlay.

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Mixing ratio liquid and powder:

125 g + 30 ml

- Separate the muffle with a thin layer of Vaseline and place the Wax Spacer.
- The plunger channel should not contain any investment residue.
- Use an undulating bottom plate in the burnout furnace to avoid cracks in the investment.
- The ceramic pellets should lay flat in the plunger channel.
- Important: Be aware of the position of the thermal element. Place the muffle at the edge of the ceramic furnace, if necessary.
- Stop the pressing time after 90 seconds.

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100 g + 24 ml

200 g + 48 ml

- The ringformer paper is suitable for 100 g and 200 g muffles.
- If a 100 g muffle is used, superfluous Ringformer Paper should be cut away.
- Make sure that the Topshaper is applied correctly.
- If the Ringformer Paper was used, remove the seam.
- Avoid polishing the underside with a trimmer, use sandpaper.
- Use an undulating bottom plate in the burnout furnace to avoid cracks in the investment.
- The ceramic pellets should lay flat in the plunger channel.

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Elephant Dental B.V.