# CARRARA Paint OUTSIDE ONLY

Drying time/Trocknen	Starting temp./Starttemp.	Vacuumstart at/Vakuumstart	Rate of increase/Aufheizrate	Final Temp./Endtemp.	Holding Time/Haltezeit	Appearance/Erscheinung
2-3 Min.	450°C	non/kein	55-80°C/Min.	825-830°C	1-2 Min.	Silky matt/shiny/ seidenmatt, glänzend

For the imitation of caries effects. Silky matt! Medium grained. Rests on top, infiltrates weakly.

For the imitation of plaque effects. Silky matt - matt! Medium grained. Rests on top, infiltrates weakly. Thin application only.

For the imitation of chalk spots. Silky matt - shiny! Medium to fine grained. Rests on top, infiltrates weakly. Can also be used to mix with incisals.

Exception! To apply on incisal edge. Remarkably stable edge after the glaze firing. Shiny!

For the imitation of smoker - coffee-tea accretion. Silky matt !! Medium grained. Rests on top, infiltrates weakly.

*Through the Outside Only Paints, you are enabled to achieve structural changes at the surface of the tooth, as they do not fully penetrate the ceramic. Effects such as coffee - tea accretion can be applied easily and removed mechanically. A true highlight is the Outside Only Halo, which is the first standard material that guarantees the correct imitation of the bright incisal junction. Stable at the edge and natural in appearance.*

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Use it in the incisal area to intensify the depth effect. Supplementary to the inside.

Use it in striking zones such as the gradual part of the occlusal elements or as a milky effect in different prominent areas.

For deshading the approximal area or to color the fissure. To intensify the character of abrasive effects or deshade the neck area.

For deshading the approximal area or to color the fissure. To intensify the character of abrasive effects or deshade the neck area.

For deshading the approximal area or to color the fissure. To intensify the character of abrasive effects or deshade the neck area.

*The Inside/Outside Paints are meant for daily laboratory use. They are reduced to the most common used colours. The limited assortment simplifies the reproduction of identical restorations even after several years. Inside/Outside Paints can be applied for both internal as well as for external purpose. The fine grained structure ensures the harmonic infiltration.*

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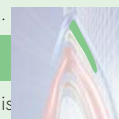
Applicable in the incisal area to intensify the depth effect. Ment to apply on the dentin care or mixed into the dentine. Fine grained.

Is ment to be mixed into the dentine or the incisals to intensify the value. Can also be used to duplicate cracks. Fine grained.

Use it to intensify the chroma, especially at the canine, in the region of the neck or for the abrasive characters. Ment to be mixed into dentine. Fine grained.

Same as Orange plus used for coloring of the approximal space. Mixto achieve secondary dentine.

Can be mixed into the dentine or the incisals. To intensify a harmonic transition from red to white aesthetics.

*The Inside Only colours enclosed in the Carrara Paint System, serve as interactive, individual characterization for fluorescent effects. These Paints are merely meant for mixing or internal staining. Strong chromatic appearances (e.g. at the canine) can easily be achieved by applying small quantities of these strong fluorescent Paints. Please note, that applying a thick layer or mixing too strong can lead to bubbles in the porcelain.*

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